



SINGLE USE BULLDOG CLAMP

INDICATIONS FOR USE:

A manual surgical instrument used to grip, hold, compress or secure sutures, tissues or vessels during general surgical procedures. These instruments are disposable and intended for single use.

DESCRIPTION:

The Cobra Biomedical single use bulldog clamps provides secure gripping, holding compressing or securing of sutures, tissue and vessels. Cobra Biomedical single use bulldog clamps are designed for easy placement and removal when used in conjunction with a standard surgical grasper or forcep.

INSTRUCTIONS FOR USE

The Cobra Biomedical single use bulldog clamps is applied by pressing the finger grips between the thumb and forefinger to open the jaws, placing it over the suture, tissue or vessel, then slowly releasing finger pressure. The rear spring section of the device closes the jaws and provides atraumatic pressure. Elements of the clip are radiopaque. An alternate method for applying the clip is to use a grasper or forcep. Compression of the grasper or forcep on the finger grips opens the jaws of the clip. When the instrument is slowly released, the spring activation will close the clip jaws.

DESIGN FEATURES:

- Lightweight
- Compact design
- Atraumatic soft jaws
- Easy to apply manually or with standard surgeon's tools
- May be used for anchoring suture
- Radiopaque

HOW SUPPLIED:

The Cobra Biomedical single use bulldog clamps is supplied sterile, two (2) per package. Cobra Biomedical single use bulldog clamps are for single use only.

WARNINGS:

Do not resterilize and/or reuse the device. Resterilization and/or reuse may compromise the integrity of the device. This may lead to potential risks of failure of the device to perform as intended and/or cross-contamination associated with using inadequately cleaned and sterilized devices. Proper disposal of contaminated devices is required.

STORAGE :













Store in an area that is dry and clean, away from heat and chemical fumes.

NOTICE:

If any serious injury has occurred to the patient or user of this device, notify Coba Biomedical or appropriate authority with the details.

PRODUCT INFORMATION DISCLOSURE:

Coba Biomedical excludes all warranties, whether expressed or implied by operation of law or otherwise, including, but not limited to: Any implied warranties of MERCHANTABILITY or FITNESS, since storage and handling of this device by the user, as well as factors relating to the patient, the diagnosis, treatment, surgical therapy, and other matters beyond Coba Biomedical’s control directly affect this device and the results obtained from its use. Coba Biomedical shall not be liable for INCIDENTAL or CONSEQUENTIAL LOSS, DAMAGE, or EXPENSE directly or indirectly arising from the use of this device. Coba Biomedical neither assumes, nor authorizes any other person to assume for it, any other or additional LIABILITY or RESPONSIBILITY in connection with this device.

Symbol	Symbol Title	Explanatory Text
	Manufacturer	Indicates the medical device manufacturer.
	Date of Manufacture	Indicates the date when the medical device was manufactured.
	Use-by Date	Indicates the date after which the medical device is not to be used. Date format is YYYY-MM-XX
	Instructions For Use	Indicates the need for the user to consult the instructions for use.
	Catalog#	Indicates the manufacturer’s catalog # so that the medical device can be identified.
	Quantity	Indicates the # of unit per package
	Batch code	Indicates the manufacturer’s batch code so that the batch or lot can be identified.,
	Do not re-sterilize	Indicates a medical device that is not to be resterilized.
	Do not use if package is damaged.	Indicates a medical device that should not be used if the package has been damaged or opened.
	Sterilized using EtO	Indicates a medical device that has been sterilized using EtO.
	Prescription	The symbol for Prescription Device
	Medical Device Symbol	an indication that the device is a medical device

For more information or to place an order, please contact a Coba Biomedical distributor, or call...

<p>South Africa</p> <p>King Fisher Medical 30 Birt St, Fairmount, Johannesburg, 2192</p> <p>sales@kfmed.co.za</p> <p>+27-11-485-0058</p>	<p>The United States of America (USA)</p> <p>Wexler Surgical 11333, Chimney Rock Road Building A Houston, Texas 77035</p> <p>sales@wexlersurgical.com</p> <p>Toll-Free: 1-800-414-1076</p> <p>Phone: 713-723-6900</p> <p>Fax: 713-723-6906</p>
--	--



Unit 5, 23 President Steyn Ave
Westdene
Bloemfontein
South Africa (ZA)
9301

info@cobabiomed.co.za

Coba Biomedical™ 2022.
All rights reserved.